


EndoMAXX EVT™

Fully Covered Esophageal Stent with Reflux Control Valve

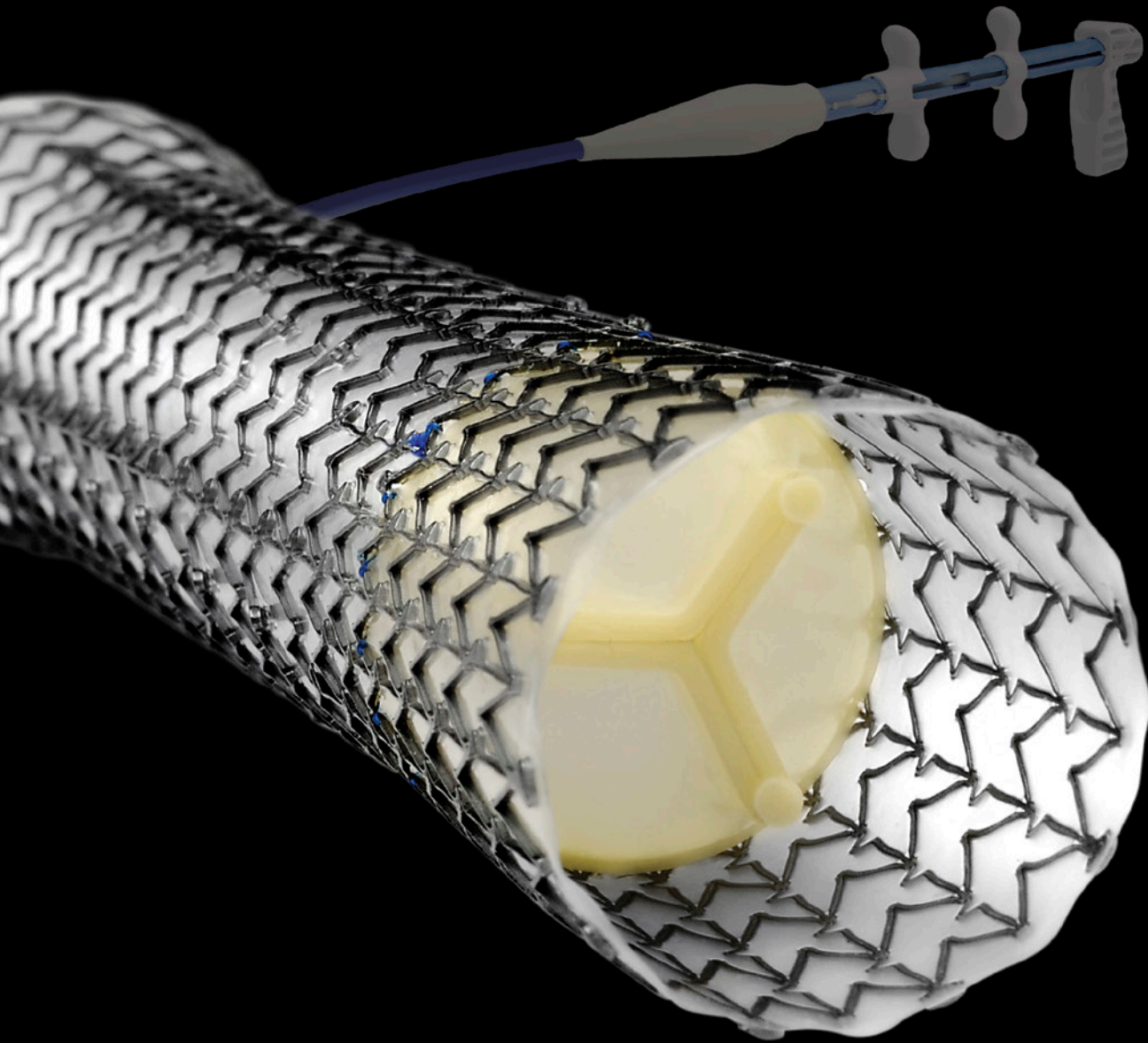

EXPANDING THE POSSIBILITIES™




MERRITMEDICA
ENDOTEK®

EndoMAXX EVT™

Fully Covered Esophageal Stent with Reflux Control Valve

The EndoMAXX EVT Fully Covered Esophageal Stent with Reflux Control Valve introduces the integration of the latest in esophageal stent and polymer valve technologies. Based on the EndoMAXX Esophageal Stent platform, the integrated reflux control valve is designed to reduce reflux, a common complication when stenting across the gastroesophageal junction.*

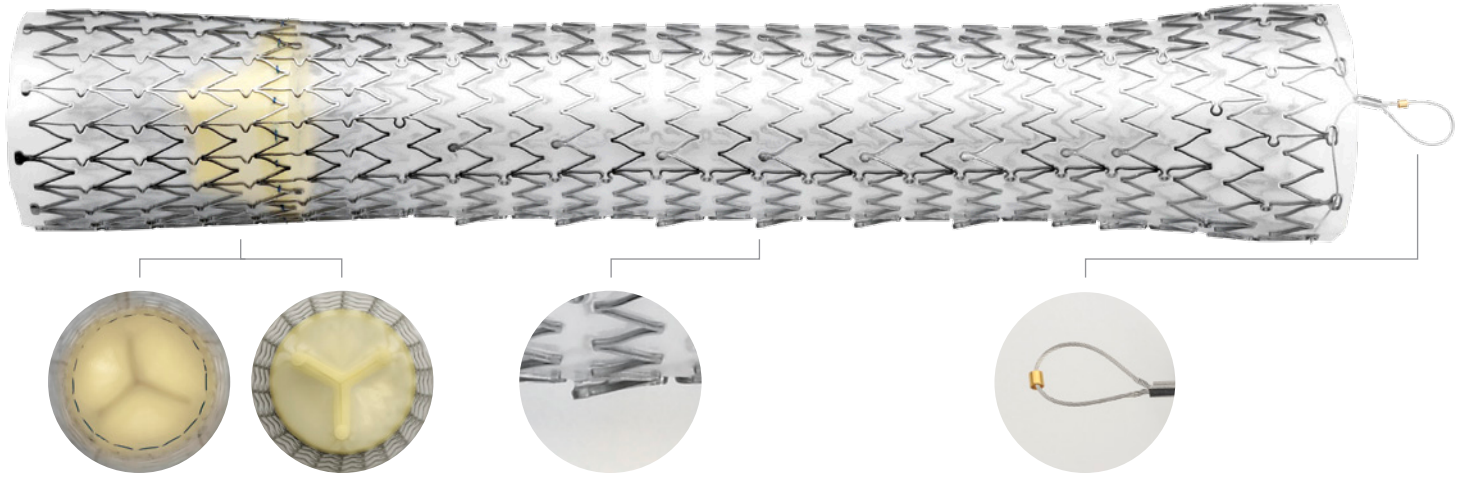

TRICUSPID VALVE

Available in 19mm and 23mm, the tricuspid valve is intended to allow the passage of food and liquids. The proprietary design characteristics will allow the valve to invert in the course of belching and/or vomiting and revert back to a closed position to mitigate reflux.*

Retrograde pressure ≥255 mm H₂O, <550 mm H₂O

Antegrade flow >100 mL/min

Valve return pressure ≤133 mm H₂O


**Based on Merit Medical Endotek Bench Testing*

ANTI-MIGRATION STRUTS

The proprietary anti-migration struts are fully covered and intended to mitigate migration by providing increased luminal wall contact.

METAL SUTURE

The proximal metal suture with gold marker provides excellent suture integrity with increased endoscopic and fluoroscopic visibility for stent repositioning and/or removal.

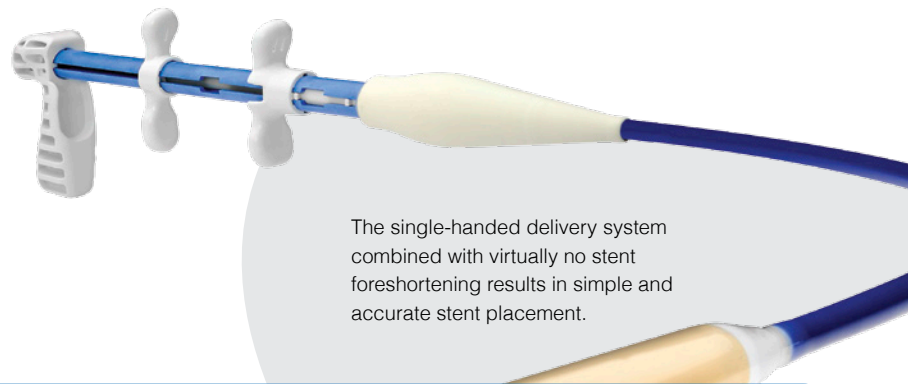

The single-handed delivery system combined with virtually no stent foreshortening results in simple and accurate stent placement.

Not available for sale in the U.S.

EndoMAXX EVT™ Fully Covered Esophageal Stent with Reflux Control Valve

Catalog Number	Stent Size Options				Delivery Device			Packaged
	Mid-Body Stent Diameter (mm)	Stent Length (cm)	Proximal Stent Flare Diameter (mm)	Distal Stent Flare Diameter (mm)	Catheter Diameter (F)	Stent Pod Diameter (F)	Working Length (cm)	
MAXX-V-1910EU	19	10	24	22	18	37	70	1 Unit
MAXX-V-1912EU	19	12	24	22	18	37	70	1 Unit
MAXX-V-1915EU	19	15	24	22	18	37	70	1 Unit
MAXX-V-2310EU	23	10	28	26	18	37	70	1 Unit
MAXX-V-2312EU	23	12	28	26	18	37	70	1 Unit
MAXX-V-2315EU	23	15	28	26	18	37	70	1 Unit



Merit Medical Systems, Inc. • 1600 West Merit Parkway • South Jordan, Utah 84095 • 1-801-253-1600 • 1-800-35-MERIT

endotek.merit.com

Merit Medical EUROPE, AFRICA AND THE MIDDLE EAST (EMEA) Amerikalaan 42, 6199 AE Maastricht-Airport, The Netherlands Tel: +31 43 358 82 22

Merit Medical Ireland Ltd. • Parkmore Business Park West Galway, Ireland • +353 (0) 91 703 733

Free phone for specific country: Austria 0800 295 374 • Belgium 0800 72 906 (Dutch) 0800 73 172 (French) • Denmark 80 88 00 24 • France 0800 91 60 30

Germany 0800 182 0871 • Ireland (republic) 1800 553 163 • Luxembourg 8002 25 22 • Netherlands 0800 022 81 84 • Sweden 020 792 445 • UK 0800 973 115

403067001/A ID 092413